



## Step 1 (if M-2443 / M-2483 are available)

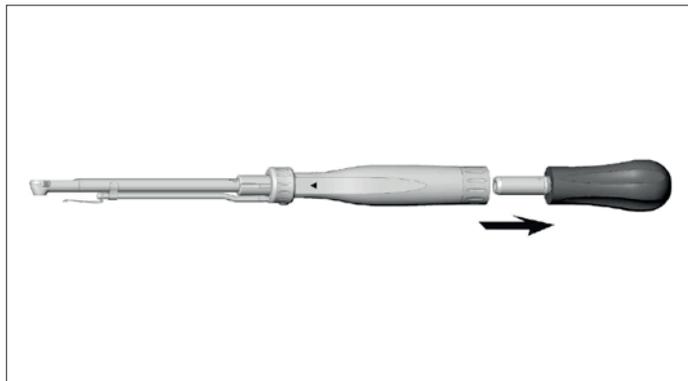


Remove the screwdriver's rotation knob

## Step 2 (if M-2442 is available)

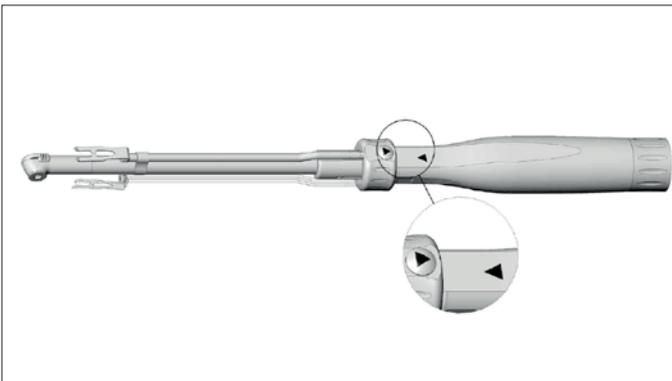


Turn the screwholding fork into the unlocking position

## Step 3 (if M-2442 is available)

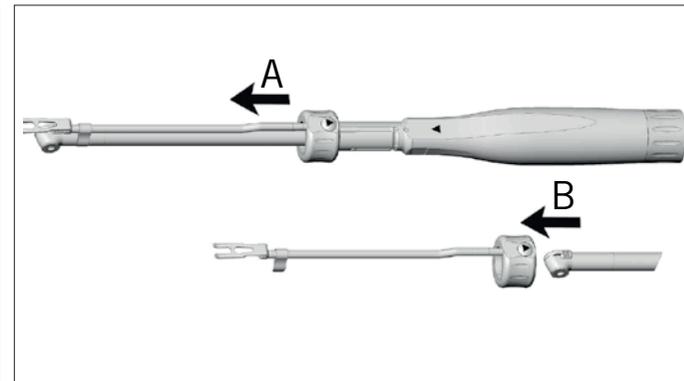


**A** Push the screwholding fork forwards until the clip releases  
**B** Remove the screwholding fork in a forwards direction

## Step 4

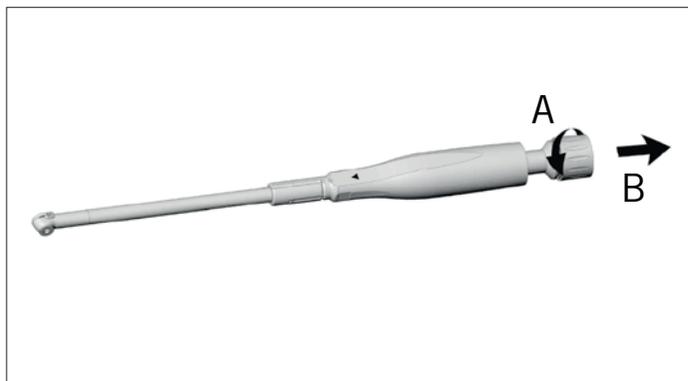


**A** Screw on the shaft  
**B** Remove the shaft from the sheath

## Step 5

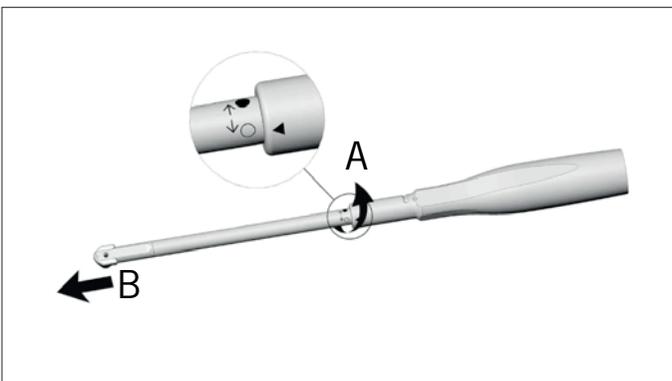


**A** Loosen the head from the sheath by turning it from «●» to «○»  
**B** Remove the sheath from the head

## Step 6

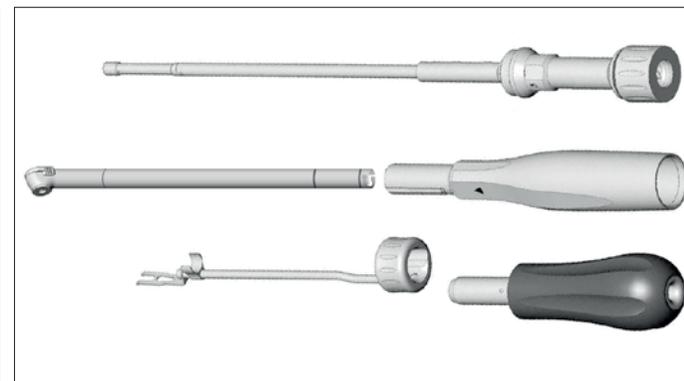


Disinfection and cleaning

## Step 1 (if M-2443/M-2483 are available)

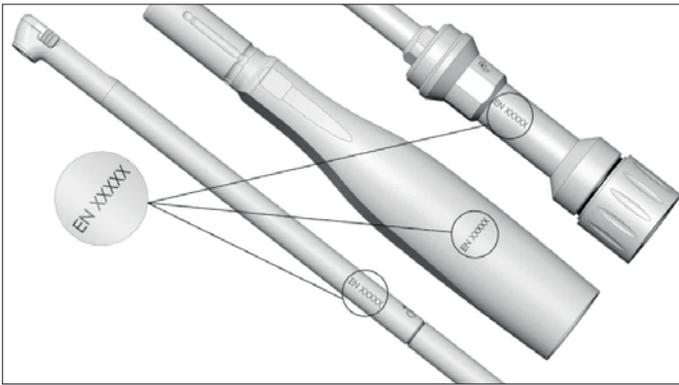


Check that the EN number on the sheath, head and shaft match

## Step 2 (if M-2442 is available)

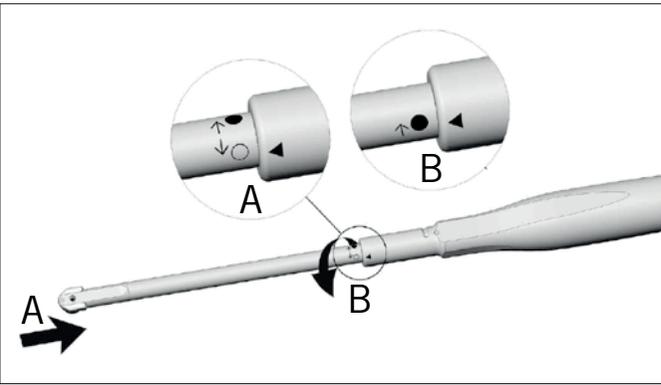


**A** Insert the head into the sheath until it engages  
**B** Turn the head from «○» to «●» until it locks

## Step 3 (if M-2442 is available)

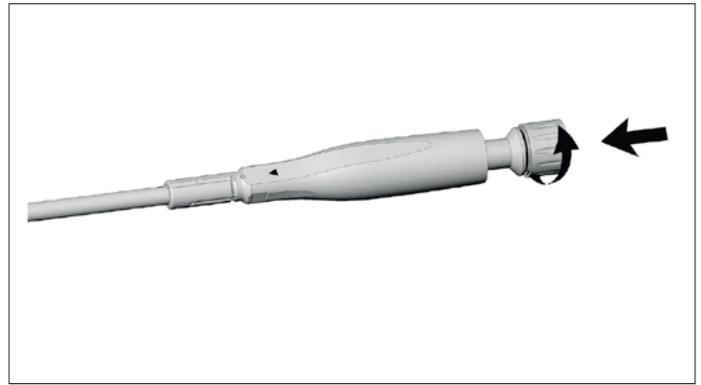


Push the shaft into the sheath and screw this tightly in place

## Step 4

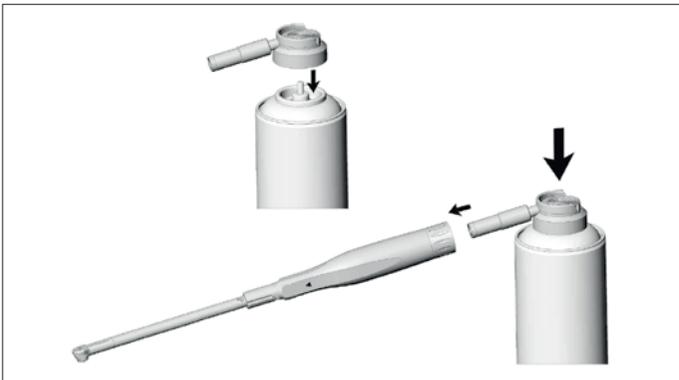


Oil service – After internal cleaning, before each sterilization

## Step 5 (if M-2442 is available)

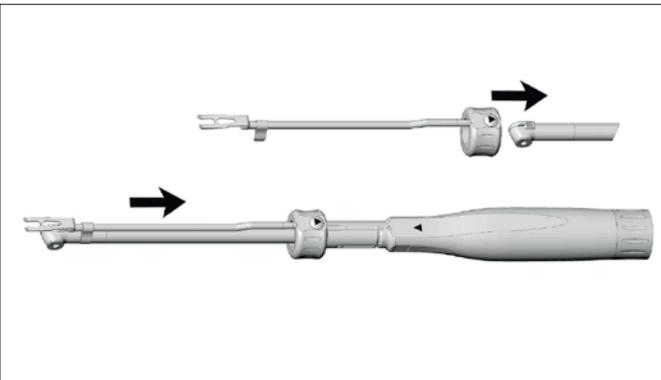

Push the screwholding fork until it audibly engages

## Step 6 (if M-2442 is available)

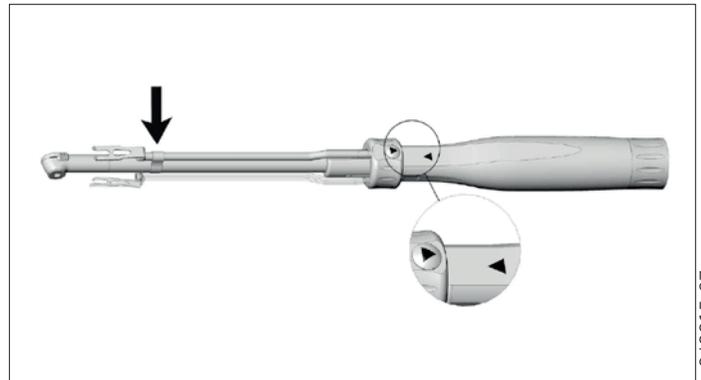

Pay attention to the positioning and tighten the clip